

## ***The Value of Autogenous***

*Simple. Natural. Efficient.*




- ***Contains bone in a sterile environment, limiting patient contamination.***
- ***Simplifies graft collection and reduces operating time and post-operative patient discomfort.***
- ***Disposable Safescraper reduces the risk of cross contamination associated with reusable products.***





**Safescraper®**

## The Simple Way To Natural Value

The Safescraper's® contoured shape provides a comfortable grip. Hold the Safescraper with the transparent chamber facing the bone. Collect the bone by pressing the Safescraper towards the bone surface while pulling backwards.

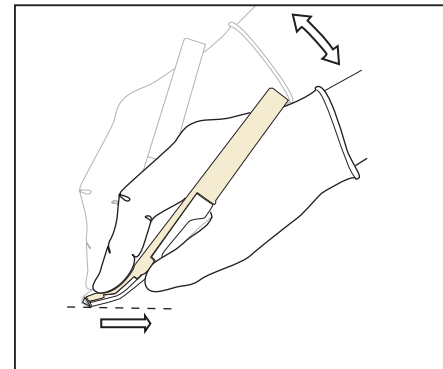

To deposit the collected bone, hold the Safescraper with transparent chamber facing down and slide the blade handle back. Use a spoon to transfer bone into the site.

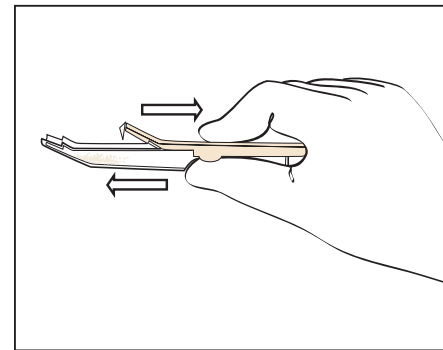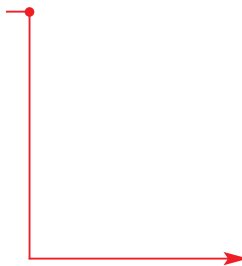

**Safescraper draws bone volumes up to 5cc**

### Ordering Codes

3598	Safescraper Straight	3 units
3987	Safescraper Curve	3 units

Distributed by:  
 Implant Innovations, Inc.  
 Global Headquarters  
 4555 Riverside Drive  
 Palm Beach Gardens, FL 33410  
 1.800.342.5454  
 Outside U.S.: + 561.776.6700  
 Fax: + 561.776.1272  
 www.3i-online.com



ART819  
 REV B 07/03

### SUBSIDIARIES

**BRAZIL**  
 Phone: +55-11-6605-3434  
 Fax: +55-11-6605-8557

**CANADA**  
 Phone: +514-956-9843  
 Fax: +514-956-9844

**MEXICO**  
 Phone: +52-55-5531-1125  
 Fax: +52-55-5203-0949

### DISTRIBUTORS

**ARGENTINA**  
 Dentalmax, SA  
 Phone: +541-1482-71001  
 Fax: +541-1482-67373

**CHILE**  
 Cybel, SA  
 Phone: +56-2-2321883  
 Fax: +56-2-2330176

**COLOMBIA**  
 Implantes y Componentes  
 Phone: +571-612-9362  
 Fax: +571-612-1542

**COSTA RICA**  
 Implantec S.A.  
 Phone: +506-2-256411  
 Fax: +506-2-247620

**EL SALVADOR**  
 Dentimerc SA de CV  
 Phone: +503-263-6350  
 Fax: +503-263-6676

**PANAMA**  
 Odontomedica, S.A.  
 Phone: +507-2-239622  
 Fax: +507-2-239621

**PARAGUAY**  
 Andres H. Arce y Cia SRL  
 Phone: +595-21-208185  
 Fax: +595-21-496291

**URUGUAY**  
 Jelenko Distribution SRL  
 Phone: +598-2-4083003  
 Fax: +598-2-4034163

3i and design are registered trademarks of Implant Innovations, Inc.  
 Safescraper is a registered trademark of C.G.M. S.p.A Divisione Medica META  
 ©2003 Implant Innovations, Inc. All rights reserved.